



2010-10-19

To whom it may concern

This letter will not indemnify the “S-CUT Instruction of use”

Handle your S-CUT

To be sure of best performance of your S-CUT it is important the cutting blade is sharp.

The cutting blade is stationary when cut, which means that only a small sector of the cutting blade been used.



When cutting in lighter clothes, t-shirt, jeans or such material one sector of the cutting blade can be used for several layers of fabric.

Test showing one sector can be used for as many as 30 patients, in above mentioned type of clothes.

When cutting motorcycle dress or other heavier material, you need rotate the cutting blade after each patient.

Rotate the cutting blade regularly to ensure you always have a sharp cutting blade ready for use in emergency. See video

S-CUT Emergency cutting tool Instruction Ger

http://www.youtube.com/watch?NR=1&v=Wj_A6ercUuE

S-CUT Emergency cutting tool Instruction Eng

<http://www.youtube.com/watch?v=fTHs-rM817s&feature=related>

Cleaning

Disassemble the cutting function before cleaning S-CUT.

Clean S-CUT in soap water solution and then with 70% alcohol.
If soaked with blood use Virkon or similar.

Dry before assemble the S-CUT again.

ES Equipment AB

ES Equipment AB
Industrivägen 1
S-449 44 Nol
Sweden

Cell phone: +4676 76 10 49 682

www.esequipment.se

www.s-cut.us.com

kontaktaoss@esequipment.se